

Range of Electromedical Devices

MEDTECH
Caring for life



NEBULIZER
HANDYNEB-GOLD



NEBULIZER
HANDYNEB-PLUS



NEBULIZER
HANDYNEB-SMART



NEBULIZER
HANDYNEB-CLASSIC



NEBULIZER KIT
NEBUKIT



OXYGEN CONCENTRATOR
OXYTEC-SMART



STEAM INHALER
HANDYVAP VAP-01



PULSE OXYMETER
OXYGARD OG-05



AUTO CPAP
HANDYPAP



DIGITAL THERMOMETER
HANDY TMP-02



INFRARED THERMOMETER
FE-03



DIGITAL THERMOMETER
HANDY TMP-03



BP MONITOR
BP-09N



INFRARED THERMOMETER
F-05



AIR PRESSURE MATTRESS
AIRBED AB-03



LED BP MONITOR
MERCURY FREE BP MONITOR



AIR PURIFIER
AIROKLEEN AP-01



NEEDLE BURNER & SYRINGE DESTROYER
MEDTECH DOTS

MEDTECH
Caring for life

HANDYPAD

HEATING PAD HP-01



UNIVERSAL SIZE

IMPORTANT SAFETY INSTRUCTION :

NOTE: TO REDUCE RISKS OF BURNS, ELECTRIC SHOCK, FIRE, AND PERSONAL INJURY, THIS PRODUCT MUST BE USED IN ACCORDANCE WITH THE FOLLOWING INSTRUCTIONS

Read all Instructions.

- Do not use if wet.
- Do not use while sleeping.
- Do not use the cord as a handle.
- Do not use in an Oxygen atmosphere.
- Do not use on areas of insensitive skin.
- Do not use Pins or other metallic means to fasten this Pad in Place.
- Do not use on an infant or on an Animal. To Fasten this Pad in Place.
- Do not use this Pad with liniment, salve or ointment preparation, especially ones that contain Heat producing Ingredients. Skin burns could result.
- Do not tamper with this Pad in any way.
- Do not bunch, sit on or against or crush Pad-Avoid sharp folds.
- Do not bend or pinch cord and Do not use if cord is damaged.
- This Pad is not to be used on by an invalid, a sleeping or unconscious Person,
- A Person with diabetes or A Person with poor Blood circulation.
- Burns can occur regardless of control setting check skin under Pad frequently.
- This heating pad is not designed for use in hospitals.
- Never pull this pad by the supply cord.
- This unit should not be used by children without Adult supervision.
- Place Pad on Top of and not under the part of the body needing heat.Unplug when not in use.



MedTech Life
Caring for life

HANDYPAD

HEATING PAD HP-01

WARRANTY CARD

Subject to Mumbai Jurisdiction only

No. _____

Model _____

Serial No. _____

Invoice No. Date : _____

Stamp and Signature
of Dealer

Name and Address
of Dealer



IMPORTANT : Warranty is for technical defects only, Heating pad cover is not under warranty.

MFG By: MEDTECH LIFE PVT. LTD.

Regd. address: B6, Byculla Service Industries, Off D. K. Marg, Byculla, Mumbai-400 027, Maharashtra, INDIA.

For Consumer complaint, Please contact Consumer Care Officer at Regd. address  +91 720 8088 720

 support@medtechlife.com  www.medtechlife.com

