

Range of Electromedical Devices

MEDTECH[®]
Caring for life



NEBULIZER - HANDYNEB
CLASSIC



NEBULIZER - HANDYNEB
SMART



NEBULIZER - HANDYNEB
PRO



BP MONITOR - NOVACHECK
BP-18



BP MONITOR - NOVACHECK
BP-11



BP MONITOR - NOVACHECK
BP-11 BL



STEAM INHALER - HANDYVAP
VAP-01



PULSE OXIMETER - OXYGARD
OG-05



DIGITAL THERMOMETER - HANDY
TMP-02



PERSONAL SCALE
WS



MASSAGER - GUN
GMV-4



MASSAGER - GUN
MPV-1

MEDTECH[®]
Caring for life

HEATING PAD Handypad HP-11



IMPORTANT SAFETY INSTRUCTION :

NOTE: TO REDUCE RISKS OF BURNS, ELECTRIC SHOCK, FIRE, AND PERSONAL INJURY, THIS PRODUCT MUST BE USED IN ACCORDANCE WITH THE FOLLOWING INSTRUCTIONS

Read all Instructions.

- Do not use if wet.
- Do not use while sleeping.
- Do not use the cord as a handle.
- Do not use in an Oxygen atmosphere.
- Do not use on areas of insensative skin.
- Do not use Pins or other metallic means to fasten this Pad in Place.
- Do not use on an infant or on an Animal. To Fasten this Pad in Place.
- Do not use this Pad with liniment, salve or ointment preparation, especially ones that contain Heat producing Ingredients. Skin burns could result.
- Do not tamper with this Pad in any way.
- Do not bunch, sit on or against or crush Pad-Avoid sharp folds.
- Do not bend or pinch cord and Do not use if cord is damaged.
- This Pad is not to be used on by an invalid, a sleeping or unconscious Person,
- A Person with diabetes or A Person with poor Blood circulation.
- Burns can occur regardless of control setting check skin under Pad frequently.
- This heating pad is not designed for use in hospitals.
- Never pull this pad by the supply cord.
- This unit should not be used by children without Adult supervision.
- Place Pad on Top of and not under the part of the body needing heat. Unplug when not in use.



MEDTECH[®]
Caring for life

HANDYPAD

HEATING PAD HP-11

WARRANTY CARD

Subject to Mumbai Jurisdiction only

No. _____

Model _____

Serial No. _____

Invoice No. Date : _____

Stamp and Signature
of Dealer

Name and Address
of Dealer



IMPORTANT : Warranty is for technical defects only, Heating pad cover is not under warranty.

Manufactured By

MEDTECH LIFE PRIVATE LIMITED

Regd. address : B6, Byculla Service Industries, D.K.Marg, Sussex Road, Byculla (East), Mumbai 400027, Maharashtra, INDIA.
Customer Care No. +917208088720
E-Mail: support@medtechlife.com
Website : www.medtechlife.com

Scan QR code for
Repair & Service
within India only
Free Factory
Free Minimum
Warranty & Replacement



Check out our
wide range of
products

