
HANDYVAP
VAP-01
STEAM INHALER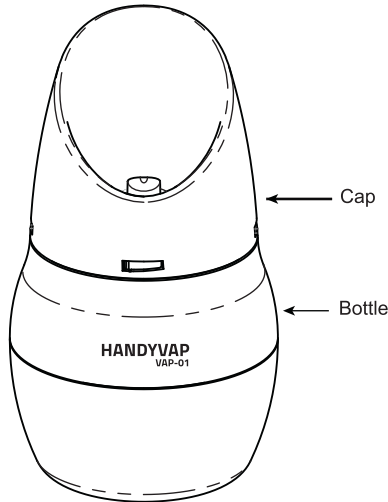

Features:

- 100% Plastic body to prevent shocks
- Cord 1.5 Meter Length
- Double wall body to avoid any injury due to hot water while handling the machine.
- Special Locking to avoid accidental opening to safeguard from spilling hot water and burn.

Few tips for safety

Keep unit straight while in use or filled with hot water

Never leave unit unattended with kids to avoid burn injury due to hot water

Never plug in wet pins always keep it dry and away from water

Don't pull wire to unplug machine, switch off & catch the plug and pull out.

CAUTION

- Use soft drinking water only.
- Remove plug from power outlet before opening to prevent risk of electric shock.

Please Note: Picture for illustration purpose only Actual product and accessories may vary.

History:

Steam has been used for hundreds of years as a therapy for treating congestion, cold and various ailments of the nose, throat and upper respiratory area.

Steam inhalers today have very convenient, it instantly create steam that can be inhaled for the best effect.

Steam inhalers are very easy to use. They have a small plastic receptacle for water and need to be plugged in to instantly produce steam .

Doctors all over the world recommend steam as a natural decongestant for anyone who is suffering from congestion, cold or having trouble with sinusitis.

A steam inhaler essentially gives off a fine mist of steam that easily reaches the inner areas of the nose, sinuses and the throat. This steam provides effective and quick relief to the person who inhales it.

The steam when it reaches the upper respiratory system relieves inflammation and congestion. It moistens dry nasal passages, and thins and loosens thick mucous, so that the congestion gets cleared more easily. When you inhale the steam it also relaxes the muscles and relieves coughing to an extent.

A Steam inhaler provides instant steam and is a safe, drug free and convenient way give fast and natural relief people suffering from colds, congestions and sinus congestion, laryngitis or allergies. It provides the right amount of humidity where it is required in the upper respiratory area.

Steam inhalers are very effective for relieving the inflammation and congestion of your sinuses, relieving the symptoms of dry mucous membrane and keeping your nasal passages moist and clean.

Useful for:

Useful for steam inhalation for treatment of :

- Cold • Bronchitis • Laryngitis.
- Sinus • Asthama
- Aroma Therapy & some Ayurvedic treatment.

How To Use:

- Open the cap by moving it anti-clockwise.
- Fill the bottle upto half level & move it clockwise to close the cap.
- Put the plug in socket & switch on power, wait for some time.
- Vapor start coming uniformly, then take your nose close to vapor and inhale.
- When finish inhalation switch off the power.
- Ensure the water level is not below minimum level.

Technical Features:

Electrical Requirements	220 - 240 V ~ 50/60Hz
Electrical Shock Protection	Class II, Type B
Dimensions // Weight	100mm x 100mm x 180mm, 250g approximate
Operating requirements	Temperature: min. +5° C – max 40°C, Humidity: 15 – 95% RH non condensing Pressure: 700 hPa, max 1060 hPa
Ingress protection	IP21
Storage Temperature Range	Temperature: min. -20° C – max 70°C, Humidity: 5 – 95% RH Non-Condensing Pressure: 700 hPa, max 1060 hPa

WARNING :

Although safe to use, it is not recommended to use the device with very hard water (non-drinkable). It will produce excessive (practically un-bearable) steam.

Mfg. By

MEDTECH LIFE PRIVATE LIMITED

Regd. address : B6,Byculla Service Industries, D.K.Marg, Sussex Road, Byculla (East), Mumbai 400027, Maharashtra, INDIA.

Customer Care No.+91 72080 88720
E-Mail:support@medtechlife.com
Website : www.medtechlife.com

Plot No. 05, Morai Industrial Park, S.No. 315/5, Morai,Valsad, Gujarat-396185, INDIA

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