


Steam Inhaler Handyvap VAP-200

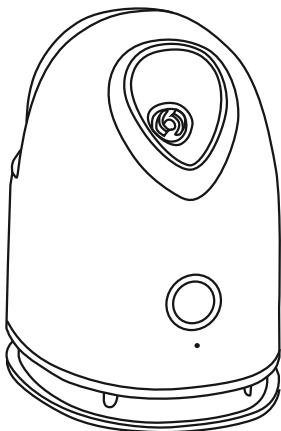

Instruction Manual

PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY

SAFETY WARNING

To avoid injury please read the following instructions. This will help you properly use this product.



CAUTION: HOT!

- Keep away from children.
- Always keep 8 inches away from steam nozzle.
- Place on a flat surface when in use. Do not hold in hands.
- As with any electrical device, use caution when operating near areas of water such as a sink or tub.
- Unplug after each use and keep unplugged while not in use.
- Be sure the product is fully plugged in during each use.
- Do not use wet hands to unplug.
- Do not use this product near any flammables such as alcohol, gasoline, acetone, sprays, etc.
- Do not immerse this product directly into water.
- Do not use product if cable or plug is damaged or hot to the touch.
- Do not use product if the plug on the extension cord becomes loose or the cord is twisted or distorted.
- Do not store in a humid area. Store in a dry, temperature controlled room.
- Do not use if anything is blocking the steam nozzle.
- Do not attempt to disassemble or repair this product on your own.
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision. (For EU market)
- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance. (For market other than EU)
- After use the appliance should be cleaned to avoid the accumulation of grease and other residues.
- For additional protection, the installation of a residual current device (RCD) having a rated residual operating current not exceeding 30 mA is advisable in the electrical circuit supplying the bathroom. Ask your installer for advice. If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard.
- When the appliance is used in a bathroom, unplug it after use since the proximity of water presents a hazard even when the appliance is switched off.

SAFETY WARNING CONTINUED :

- Do not use more than once per day. Each time should be limited to 10 minutes. Overuse may result in expansion of pores.
- For normal skin, 2-3 times per week is suggested. For sensitive skin, once per week.
- If any kind of discomfort occurs, please stop using immediately.
- After use, allow water to cool down for 10 minutes, and then dispose of any remaining water.

NOT TO BE USED BY THE FOLLOWING :

- Women who are pregnant or menstruating, This may cause hormone disorder thereby resulting in related skin issues.
- If you have damage to your skin such as: dermatitis, sensitive skin, sunburned skin, eczema, or swelling.
- If you have recently had a medical procedure on the facial area.

SPECIFICATIONS :

Voltage	: 220-240V~, 50-60Hz
Power	: 280W
Steam temperature:	20cm above the entrance of nozzle is approximately 40°C, 104°F

PREPARATION :

When the appliance is used for the first time, some water may remain around the water tank. This is residual water used for inspection in the factory, and does not cause any sanitation problems.

1. Pull container out.
2. Remove cap.
3. Fill with distilled water
Twist cap back on



4. Place container back into steamer.
5. Plug in unit.
6. Press switch to the light on and wait approximately 50 seconds steam to release.

● Step 1

Place the facial steamer on a flat surface and plug into outlet.

● Step 2

Open the lid and fill distilled water into the tank until full. Place lid back into the body until edges are well inserted.

Note: Do not mix any impurities into the water, such as liquid soap, aroma oil, or perfume as it may damage the equipment and cause a malfunction.

● **Step 3**

Press the switch button to light on. Once turned on, allow 1-2 minutes for water to warm and begin steaming. Steam will last for 15-20 minutes depending on the amount of water placed in the tank. The water level in the tank will gradually decrease.

● **Step 4**

Once complete, Press the switch to light off and unplug the unit.

AFTER USE :

After each use, unplug power cord. Allow product to cool for 10 minutes. When cooled, remove all remaining water. Allow product to dry before storing. Product should be stored in a dry, non-damp area.

Warranty conditions :

Stem Inhaler Handyvap VAP-200 is made under strict quality control & utmost care is taken to ensure long, trouble free service.

- This unit is warranted for a period of 12 month from the date of purchase.
- We undertake to repair or replace at our discretion any part which was originally defective, provided the unit was not tampered with or mishandled.
- The warranty is only against manufacturing defects & shall not be applicable in case of damage due to accident, acts of nature, loss of components, intentional damage, and improper installation or if it was opened or service by authorized persons.
- The warranty is void, if the set is not operated according to instructions given in the "Instruction manual booklet", if any repair work is carried out by persons other than authorized company personnel, defects are caused by improper or reckless use, which shall be determined by company personnel.
- Warranty does not cover accessories external to the equipment, defects caused due to causes beyond control like lightning, abnormal voltage, acts of God or while in transit to service centre or to purchasers/premises.



**Steam Inhaler Handvav
VAP-200**

WARRANTY CARD

Subject to Mumbai Jurisdiction only

No.

Model

Serial No.

Invoice No. Date:

Stamp and Signature
of Dealer

Name and Address
of Dealer



Imp. & Mkt by: **MEDTECH LIFE PVT. LTD.**

Regd. address: B6, Byculla Service Industries,
Off D. K. Marg, Byculla, Mumbai-400 027, Maharashtra, INDIA.

For Consumer complaint, Please contact Consumer Care
Officer at Regd. address 📞/📧 +91 720 8088 720

📧 support@medtechlife.com 🌐 www.medtechlife.com