

Range of Electromedical Devices

MEDTECH[®]
Caring for life



NEBULIZER - HANDYNEB
CLASSIC



NEBULIZER - HANDYNEB
SMART



NEBULIZER - HANDYNEB
PRO



BP MONITOR - NOVACHECK
BP-18



BP MONITOR - NOVACHECK
BP-11



BP MONITOR - NOVACHECK
BP-11 BL



STEAM INHALER - HANDYVAP
VAP-01



PULSE OXIMETER - OXYGARD
OG-05



DIGITAL THERMOMETER - HANDY
TMP-02



HEATING PAD - HANDYPAD
HP-01



MASSAGER - GUN
GMV-4



MASSAGER - GUN
MPV-1


MEDTECH[®]
Caring for life

Nebulizer Kit **3X**
NebuKit[®]



Insist on

TWO PARTS BOTTLE

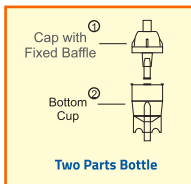

Because of fix baffle,
it is Safe for nebulization.
No fear of losing baffle

THREE PARTS BOTTLE



It is easy to loose
Baffle as it is not fixed
So if baffle falls Nebulization
cannot take place and kit is non
functional

RESPIRATORY THERAPY PRODUCT

- Disposable for Single Patient Use
- Non Intended for Reprocessing

Features:

- Non-Toxic PVC Tube and mask
- 2 Meter PVC Tube
- BPA-free, 2-Part Medicine Cup

⚠ CAUTION:

- Ensure that all connection are secured and air circulates freely through the tubing.
- Do not store at extreme temperature and humidity. Store in cool & dry place.

- Do not attach the outlet of the nebulizer directly to an endotracheal tube, tracheostomy tube or other device that does not offer an alternate pathway for exhalation.

INSTRUCTION FOR USE:

These directions are general guidelines intended for use by qualified medical personnel.

1. Place prescribed solution into the bottom cup of nebulizer bottle.
2. Attach cap to nebulizer cup with a clockwise rotation.
3. Attach the tubing between the gas source and the nebulizer.
4. Connect the nebulizer outlet on top of nebulizer cap to the mask inlet.
5. Start gas flow Check for aerosol mist; if necessary, tap the device until it begins to nebulize.
6. Place the mask on the patient's face with the elastic strap below the ears and around the neck. Gently pull the ends of the strap until the mask is secured. Shape the metal strap on the mask to fit the face.
7. During treatment, periodically tap nebulizer to minimize residual volume.

Mfg. By

MEDETECH LIFE PRIVATE LIMITED

Regd. address : B6, Byculla Service
Industries, D.K.Marg, Sussex Road, Byculla (East),
Mumbai-400027, Maharashtra, INDIA.
Customer Care No. +917208088720
E-Mail: support@medtechlife.com
Website : www.medtechlife.com

Check out our
wide range of
products

